

VDW.GOLD® - Upgrade Instructions (RECIPROC® software R 1.0)

This RECIPROC® Upgrade Stick is only available for VDW.GOLD® devices with a serial number = or > than G 9518.

VDW.GOLD® - Front view: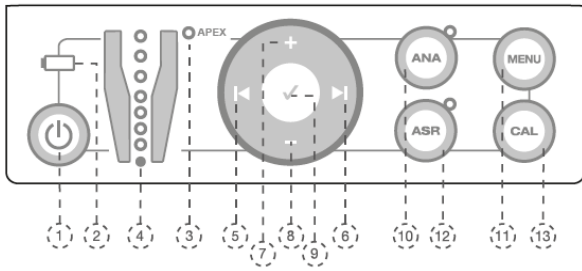

VDW.GOLD® - Rear view:

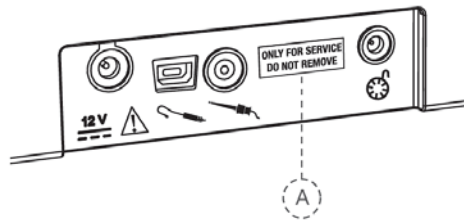Warnings:

- After the Upgrade, the serial number must be entered for verification. In case the serial number has been entered incorrectly for more than two times, the RECIPROC® Upgrade software will be erased and the latest VDW.GOLD® software version (3.5) will be saved on the device, even if the software version primarily saved on the device was older. You will now have to re-start the Upgrade process.
- The RECIPROC® Upgrade will reset your VDW.GOLD® endo motor, meaning the device will return to its factory default parameters. This reset will also include deletion of all individual Dr's Choice settings.

Initial steps prior to upgrade

STEP 1	Make sure that the battery is fully charged. Switch the device off and unplug all cables, such as the cables for micromotor, power and foot control.
STEP 2	Remove the USB protection sticker "ONLY FOR SERVICE – DO NOT REMOVE" (A) from the rear side of the device. Do not trash the sticker, as re-use is recommended after the upgrade. You are now ready to start the upgrade.

Upgrade process

Step	Action
Automatic upgrade (STEP 1)	Insert the RECIPROC® Upgrade Stick into the USB slot , then press CAL (13) and Power (1) simultaneously for 3 seconds. The automatic upgrade will start with the display message "Erasing old software" and will finalize with "Loading new software ended" . The device will switch off automatically. Please switch it back on, <u>not yet removing the Upgrade Stick</u> .
CHECK of serial number (STEP 2)	Look for the serial number at the bottom of your VDW.GOLD® device. To enter the serial number, use the arrow keys (5) and (6) to select the position as indicated on the graphic. Now select the respective digit by pressing the „+“ (7) and „-“ (8) keys until the correct digit is displayed. <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="border: 1px solid black; padding: 5px; text-align: center;"> Select serial ↓ number 00000 </div> <div style="border: 1px solid black; padding: 5px; text-align: center;"> Select serial ↓ number 07610 </div> </div> After having entered the complete five-digit serial number, confirm the number with the confirmation key (9) .
Finish the upgrade (STEP 3)	The RECIPROC® software will be transferred to your VDW.GOLD® device. The display will show the message „Update done“ and the new software version will appear on the welcome screen. You can now remove the Upgrade Stick and re-apply the sticker to cover the USB slot.

Troubleshooting

Problem	Solution
Entered wrong serial number	You can enter the wrong serial number for up to two times. After the third incorrect entry, the RECIPROC® Upgrade software will be erased and the latest VDW.GOLD® software version (3.5) will be saved on the device, even if the software version primarily saved on the device was older. You now have to restart the Upgrade process.
Display shows „Software not present on stick“	The RECIPROC® Upgrade Stick <u>can only be used for one VDW.GOLD® device</u> . If an upgrade is carried out with an already used Upgrade Stick, the display will show „Software not present on stick“ <u>and the device will not be upgraded</u> .
Any other unexpected problem	Please call the VDW service hotline: +49 (0)89 62734-555

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