

VDW.SILVER® RECIPROC® Upgrade Instructions

This RECIPROC® Upgrade Stick is only available for VDW.SILVER® devices with a serial number = or > than S 7603.

VDW.SILVER® - Front view: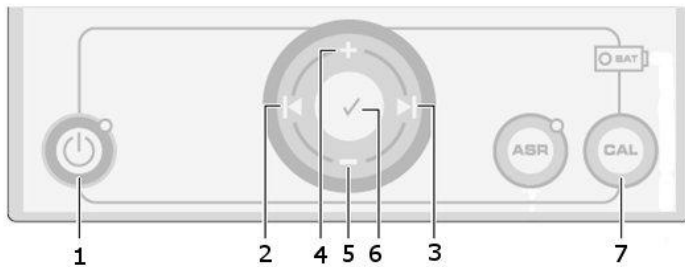

VDW.SILVER® - Rear view:

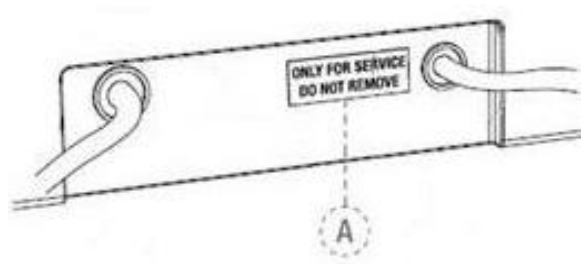Initial steps prior to upgrade

STEP 1	Make sure that the battery is fully charged. Switch the device off and unplug all cables, such as the cables for micromotor, power and foot control.
STEP 2	Remove the USB protection sticker "ONLY FOR SERVICE – DO NOT REMOVE" (A) from the rear side of the device. Do not trash the sticker, as re-use is recommended after the upgrade. You are now ready to start the upgrade.

Upgrade process

Step	Action
Automatic upgrade (STEP 1)	Insert the RECIPROC® Upgrade Stick into the USB slot , then press CAL (7) and Power (1) simultaneously for 3 seconds. The automatic upgrade will start with the display message "Erasing old software" and will finalize with "Loading new software ended". The device will switch off automatically. Please switch it back on, <u>not yet removing the Upgrade Stick</u> .
CHECK of serial number (STEP 2)	Look for the serial number at the bottom of your VDW.SILVER® device. To enter the serial number, use the arrow keys (2) and (3) to select the position as indicated on the graphic. Now select the respective digit by pressing the „-“ (5) and „+“ (4) keys until the correct digit is displayed. <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="border: 1px solid black; padding: 5px; text-align: center;"> Select serial ↓ number 00000 </div> <div style="text-align: center;"> Example: ↓ </div> <div style="border: 1px solid black; padding: 5px; text-align: center;"> Select serial ↓ number 07610 </div> </div> <p>After having entered the complete five-digit serial number, confirm the number with the confirmation key (6).</p>
Finish the upgrade (STEP 3)	The RECIPROC® software will be transferred to your VDW.SILVER® device. The display will show the message „Update done“ and the new software version will appear on the welcome screen. You can now remove the Upgrade Stick and reapply the sticker to cover the USB slot.

Troubleshooting

Problem	Solution
Entered wrong serial number	You can enter the wrong serial number for up to three times. After that, the old software will be loaded to the device and you will have to restart the update.
Display shows „Software not present on stick“	The RECIPROC® Upgrade Stick <u>can only be used for one VDW.SILVER® device</u> . If an upgrade is carried out with an already used Upgrade Stick, the display will show „Software not present on stick“ <u>and the device will not be upgraded</u> .
Any other unexpected problem	Please call the VDW service hotline: +49 (0)89 62734-555

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