

Universal Restorative

Class II Restorations



1

Prepare the tooth and apply matrix band.




2

Follow the bonding procedure provided in the Instructions for Use of the adhesive system of choice.



3

Place composite up to 2mm.

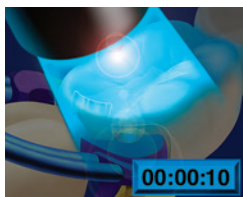

4

Light cure according to Instructions for Use.

Universal Restorative

Class II Restorations



Use contouring disk.

5



Use contouring bur.

6



Control bite registration.

7



Use finishing wheel.

8

Universal Restorative

Class II Restorations



9

Use polishing wheel.



Universal Restorative

Class IV Restorations




Prepare the tooth.

1




Follow the bonding procedure provided in the Instructions for Use of the adhesive system of choice.

2

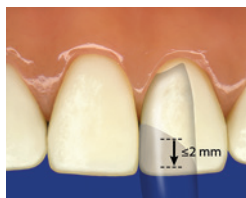

Place composite up to 2mm.

3

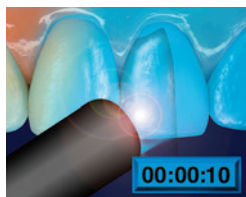


Light cure according to Instructions for Use.

4


Universal Restorative

Class IV Restorations

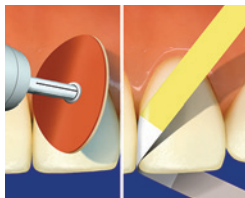


5

Use contouring disk
and finishing strip.

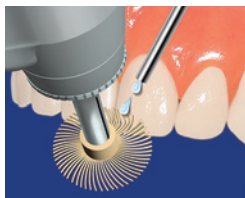

6

Use finishing wheel.

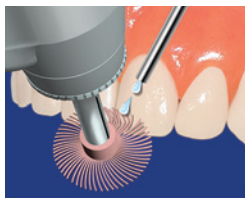

7

Use polishing wheel.

Universal Restorative

Pink Opaquer Placement




1

Prepare the tooth.




2

Follow the bonding procedure provided in the Instructions for Use of the adhesive system of choice.

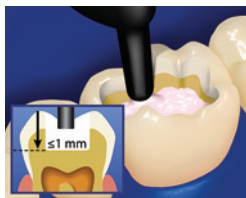

3

Place Pink Opaquer up to 1mm.

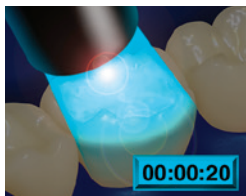

4

Light cure according to Instructions for Use.

Universal Restorative

Restoration Type	Increment Depth	Halogen Lights (with output of 550-1000 mW/cm ²)	LED Lights (with output 1000- 2000 mW/cm ²)
Class II	2.0 mm.	20 sec. Occlusal	10 sec. Occlusal
Class IV	2.0 mm.	20 sec. Incisal	10 sec. Incisal
Pink Opaquer	1.0 mm.	40 sec. Occlusal	20 sec. Occlusal

3M, ESPE and Filtek are trademarks of 3M or 3M Deutschland GmbH.
Used under license in Canada. © 3M 2019. All rights reserved.



3M ESPE
Dental Products
2510 Conway Avenue
St. Paul, MN 55144-1000 USA



3M Deutschland GmbH
Dental Products
Carl-Schurz-Strasse 1
D-41453 Neuss, Germany



44005086325