

EQUIA



1
Apply petroleum jelly
or GC Cocoa butter
inside the matrix



2
Use anatomical shaped
wedges for better
adaptation and
contact points



3
Use of tight rings
from sectional matrix
systems act as separator
of teeth to ensure good
contact points



4
Apply GC Cavity
Conditioner (10 sec.)
or Dentin Conditioner
(20 sec.)



5
Rinse and gently dry,
do not desiccate



6
Shake or tap.
Depress plunger



7
Insert on
Capsule Applicator.
Click once to activate



8
Mix for 10 sec.
Working time is
1 min. 15 sec.
from start of mix



9
Insert on Capsule
Applicator. Click twice
to prime capsule



10
IMMEDIATELY
Dispense within 10 sec.

GC

EQUIA



11
Pack and contour.
Avoid moisture
contamination and
dry-out



12
Ensure complete set of
EQUIA® Fil and carefully
remove the ring.
Use a probe to separate
the bond between
matrix and EQUIA® Fil



13
Final finishing after
2 min 30 sec. from
start of mix



14
Finish the restoration
by applying the
EQUIA® Coat.
DO NOT AIR BLOW



15
Light cure for **20 sec.**

Note: Steps except for 1, 2, 3 and 12 are common for Class I and II

GC



V0801