

# Tetric EvoCeram® Bulk Fill


1

Self-Etch




or

Etch & Rinse

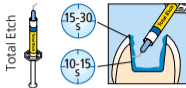

or

Selective Enamel-Etch

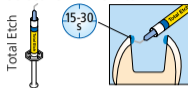

2

Adhese® Universal  
VivaPen®



or

Adhese® Universal  
Bottle



3

Tetric EvoFlow®



4

Tetric EvoCeram®  
Bulk Fill



$\geq 600 \text{ mW/cm}^2$	20 s
$\geq 1'000 \text{ mW/cm}^2$	10 s
Bluephase® Style	10 s

Bluephase® Style



ivoclar  
vivadent  
clinical



# Tetric EvoCeram® Bulk Fill

The bulk composite

Layer  
thickness



1.5 mm I  
Tetric EvoFlow  
Dentin



2.0 mm I  
Tetric EvoFlow  
Enamel



1.5 mm I  
Tetric EvoCeram  
Dentin



2.0 mm I  
Tetric EvoCeram  
Enamel



**NEW**  
4.0 mm I  
Tetric EvoCeram  
Bulk Fill



Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123



see  
Instructions

ivoclar  
vivadent  
clinical

Date information prepared: 2014-08  
639248/Rev.1